

# TESTS WITH ZOETIS/ABAXIS FUSE

## Abaxis Fuse Integration Setup

### Configuration of VetlinkPRO

The Abaxis Fuse module is not activated by default. For more information on activating this module please email [sales@cfl.co.nz](mailto:sales@cfl.co.nz).


### Configure Goods & Services

Set up your lab test products to send to Abaxis analysers. In VetlinkPRO either open an existing item or create a new item.

Set **Label Type** to **Lab Report**.

Set **User Code** to relevant test Code from Abaxis.

- The codes can be looked up in VetlinkPRO from [Options | Picklist Setup | History | Abaxis Vetscan Test Codes](#).
- New test/codes will be added by Abaxis periodically. These will automatically be downloaded from the Fuse machine into VetlinkPRO to use.
- Note** that the code must be set on the components of any kitset or surgery complex product.
- If you need to create a new product you can single click a code from the Abaxis Vetscan Test Code list, then click the add product button at the bottom of the page. This will create a new product with the Name, label Typ and Code already filled in.

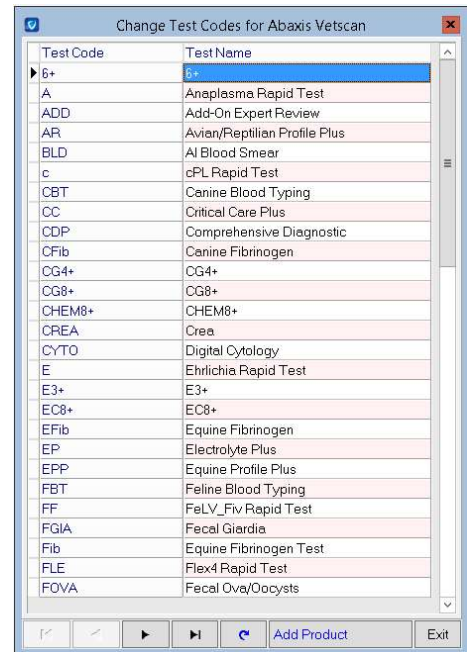

Enter other fields such as Name, category, Sell Price, etc as needed. If you wish services to have a cost price you can enable this in setup from [Options | Setup | Product | 1: Options | Allow Services to have cost price](#).

### Multiple Tests with Complex Product

Only one code can be associated with a product. Therefore if you wanted a product to fire off multiple tests, e.g. Comprehensive Diagnostic and Haematology tests, you would need to create a service complex product. Into this product you would add 2 separate component products, each containing the relevant Abaxis Test code.

## Performing Test

1. Make a bill as normal
2. Add analyser test to the bill
3. A popup windows will appear. Click **Abaxis** button, It will turn green to indicate test waiting to be sent.
4. Add any other tests.
5. **Save and Exit** bill and all tests will be sent to the analysers to process.
6. Perform tests on the analyser and they will be automatically synced back to the correct visit in VetlinkPRO.

**Lab Test Requisition**  
Choose the appropriate option

F10 - Send request to IDEXX VetLab Station	F10 - Send request to Abaxis VetSync Station	F11 - Print a lab requisition for external lab
F12 - Print lab requisition label	ESC - Close	Print Form
Print Label	Close Form	

Sub Total for Fluffy including GST: \$30.070

F11 Save/Ex	SF9 Disc Bill	Ctrl-N Notes	SF6 Rqst Sta	Quote	SF8 Remindr	STotal \$26.15
F9 New Bill	F12 Payment	SF10 Cancel	F4 Drug Lbl	SF7 Rebook	SF5 Referral	GST \$3.92
Total \$30.07						Total w ID \$30.07

**Tax Receipt** State **Open** Owes \$61.20 BP Avail : 385 \$30.07

## Endorsing New Lab Results

All new lab results imported from the Abaxis Fuse will be automatically attached to the clinical history of the patient, if the ref # from the Lab sample label has been entered in the analyser correctly. Results are also placed into the Endorse Labs repository as well.

The Endorse Labs repository is basically area that notifies users a new lab result has arrived in the system. When a report arrives the test tube icon at the top left of Vetlink will start blinking. Users can then just click the icon and check the repository to see if the result is one that they are waiting for.



If the result is one they have been waiting on then;

- **If client and patient it has attached are correct**, then click the **Endorse** button, in bottom right corner to remove it from the list.
- **If client and patient it has attached are incorrect**, because the Ref# from lab sample label was entered incorrectly in the analyser. Then click the **Move** button and select the correct client/patient and visit to link it to.